

Fast and easy maintenance- the OxyGo benefit.

OxyGONEXT™ Column Change Guide



OxyGo
Orlando, FL 32837

For service on your OxyGo, please contact
your local Equipment Provider at:

96-09242-00-01 B
1400-3425 B



OxyGo NEXT™

To change the columns on your OxyGo NEXT, first:


- 1 Turn off the OxyGo NEXT concentrator by pressing the power button to shut down the device.
- 2 Remove the OxyGo NEXT concentrator from the carrying case (if attached).




Remove the battery from the OxyGo NEXT concentrator.




Place the OxyGo NEXT concentrator on its side so that the underside is visible. The metal column assembly can be seen on one side of the device.

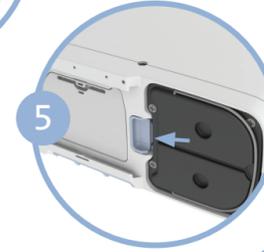

Unlock the column assembly by pushing the latch button away from the columns.

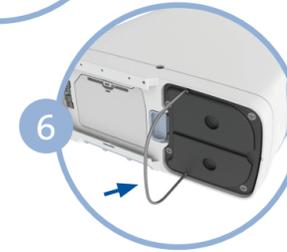

While holding the button open, slide the column assembly out of the device by pulling on the column pull handle.

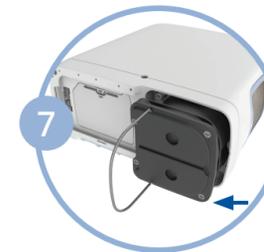

Remove the columns completely from the OxyGo NEXT. Both columns are removed as one piece.



If tape is applied to column assembly, remove tape.



Remove dust caps of new column assembly. Make sure there is no dust or debris where the dust caps were located.



Insert column assembly into the OxyGo NEXT concentrator. Do not leave the column ends exposed; column assembly should be inserted into the OxyGo NEXT as soon as the dust caps have been removed.



Push the column assembly into the device such that the columns are fully seated into the OxyGo NEXT concentrator. The spring loaded latch button should fully return to the closed position.



Connect the AC power supply cord to the OxyGo NEXT and plug the power supply AC cord into an electrical outlet. Do not power on the OxyGo NEXT concentrator.

NOTE Column change instructions are only to be used when maintenance is required and are not intended for practice purpose.



Press the alert/bell button once and screen will display "sieve reset success." Press the power button to turn on the OxyGo NEXT, and use normally.



Press and hold the plus (+) and (-) minus buttons for 5 seconds. The screen will display "sieve reset" message. Release buttons once message is displayed on screen.